



DO'S

- Use appropriate base curve to match frame curve.
- Use MEI or equivalent edging system to ensure proper beveling and to maintain frame integrity.
- Wrap compensation is required on all RX lenses mounted into a Wiley X frame.
- Only use polycarbonate lens material in a Z87 stamped frame.
- All RX lenses are required to be made to meet ANSI safety standards.
- **All Lenses Should Be Cold Snapped Into Wiley X Frames.**
- When in doubt please call the Prescription Dept @ 1.888.210.8926
- Because of the lens requirements necessary in our ANSI frames, we recommend that you only use a Wiley X authorized lab for RX'ing. RX'ing any authentic Wiley X frame through a lab that is not an authorized Wiley X lab will void your warranty. Please contact your Wiley X representative for a list of authorized labs.

DON'TS

- **Do Not Heat A Wiley X Frame.**
- Do not discard any of the information cards that come in the case with your Wiley X frames. They contain crucial lab information in order to fill prescriptions to required ANSI standards.
- Do not cut RX lenses too long or too wide, this will stress the frame causing it to crack over time.

WILEYX USA: 1.888.210.8926 // 925.243.9810 // WILEYX.COM
WILEYX EUROPE LLC: +45 96 93 00 45 // WILEYX.EU

COPYRIGHT © 2014 WILEY X, INC. ALL RIGHTS RESERVED.

WILEY X, WX, BLACK OPS, logos, tag lines and product names are trademarks or registered trademarks (marcas registradas) of Wiley X, Inc. in the U.S. and other countries. Logos © 2007-2014 Wiley X, Inc. All rights reserved.

ERX-ICRX (0)